

Table of contents

\\USER

CAMRIS

Satterthwaite

SATTERTHWAITE_ABCD_32ch

localizer_32ch
ABCD_T1w_MPR_vNav_setter
ABCD_T1w_MPR_vNav
ABCD_T2w_SPC_vNav_setter
ABCD_T2w_SPC_vNav
ABCD_fMRI_DistortionMap_PA
ABCD_fMRI_DistortionMap_AP
ABCD_fMRI_rest
ABCD_fMRI_rest
ABCD_dMRI_DistortionMap_PA
ABCD_dMRI_DistortionMap_AP
ABCD_dMRI
ABCD_fMRI_DistortionMap_PA
ABCD_fMRI_DistortionMap_AP
ABCD_fMRI_rest
ABCD_fMRI_fracnback
ASL_3DSPIRAL_V20_GE
QSM_SWI_1.5mm

\\USER\CAMRIS\Satterthwaite\SATTERTHWAITE_ABCD_32ch\localizer_32ch

TA: 0:12 PM: REF Voxel size: 0.5×0.5×7.0 mmPAT: Off Rel. SNR: 1.00 : fl

Properties

Prio recon	On
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	On
Load images to graphic segments	On
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	On
Wait for user to start	Off
Start measurements	Single measurement

Routine

Slice group	1
Slices	1
Dist. factor	20 %
Position	L0.0 A20.0 H0.0 mm
Orientation	Sagittal
Phase enc. dir.	A >> P
Slice group	2
Slices	1
Dist. factor	20 %
Position	L0.0 A20.0 H0.0 mm
Orientation	Transversal
Phase enc. dir.	A >> P
Slice group	3
Slices	1
Dist. factor	20 %
Position	L0.0 A20.0 H0.0 mm
Orientation	Coronal
Phase enc. dir.	R >> L
AutoAlign	---
Phase oversampling	0 %
FoV read	250 mm
FoV phase	100.0 %
Slice thickness	7.0 mm
TR	7.5 ms
TE	3.69 ms
Averages	2
Concatenations	3
Filter	Prescan Normalize, Elliptical filter
Coil elements	HEA;HEP

Contrast - Common

TR	7.5 ms
TE	3.69 ms
TD	0 ms
MTC	Off
Magn. preparation	None
Flip angle	20 deg
Fat suppr.	None
Water suppr.	None
SWI	Off

Contrast - Dynamic

Averages	2
Averaging mode	Short term
Reconstruction	Magnitude
Measurements	1

Contrast - Dynamic

Multiple series	Each measurement
-----------------	------------------

Resolution - Common

FoV read	250 mm
FoV phase	100.0 %
Slice thickness	7.0 mm
Base resolution	256
Phase resolution	91 %
Phase partial Fourier	Off
Interpolation	On

Resolution - iPAT

PAT mode	None
----------	------

Resolution - Filter Image

Image Filter	Off
Distortion Corr.	Off
Prescan Normalize	On
Unfiltered images	Off
Normalize	Off
B1 filter	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	On

Geometry - Common

Slice group	1
Slices	1
Dist. factor	20 %
Position	L0.0 A20.0 H0.0 mm
Orientation	Sagittal
Phase enc. dir.	A >> P
Slice group	2
Slices	1
Dist. factor	20 %
Position	L0.0 A20.0 H0.0 mm
Orientation	Transversal
Phase enc. dir.	A >> P
Slice group	3
Slices	1
Dist. factor	20 %
Position	L0.0 A20.0 H0.0 mm
Orientation	Coronal
Phase enc. dir.	R >> L
FoV read	250 mm
FoV phase	100.0 %
Slice thickness	7.0 mm
TR	7.5 ms
Multi-slice mode	Sequential
Series	Interleaved
Concatenations	3

Geometry - AutoAlign

Slice group	1
Position	L0.0 A20.0 H0.0 mm
Orientation	Sagittal
Phase enc. dir.	A >> P
Slice group	2
Position	L0.0 A20.0 H0.0 mm

Geometry - AutoAlign

Orientation	Transversal
Phase enc. dir.	A >> P
Slice group	3
Position	L0.0 A20.0 H0.0 mm
Orientation	Coronal
Phase enc. dir.	R >> L
AutoAlign	---
Initial Position	L0.0 A20.0 H0.0
L	0.0 mm
A	20.0 mm
H	0.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Sagittal

Geometry - Saturation

Saturation mode	Standard
Fat suppr.	None
Water suppr.	None
Special sat.	None

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	REF
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Sum of Squares
Save uncombined	Off
Matrix Optimization	Off
AutoAlign	---
Coil Select Mode	Off - All

System - Adjustments

B0 Shim mode	Tune up
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
A >> P	263 mm
R >> L	350 mm
F >> H	350 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Slice-sel.

System - Tx/Rx

Frequency 1H	123.255216 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	7.5 ms
Concatenations	3
Segments	1

Physio - Cardiac

Magn. preparation	None
Fat suppr.	None
Dark blood	Off
FoV read	250 mm
FoV phase	100.0 %
Phase resolution	91 %

Physio - PACE

Resp. control	Off
Concatenations	3

Inline - Common

Subtract	Off
Measurements	1
StdDev	Off
Liver registration	Off
Save original images	On

Inline - MIP

MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

Inline - Soft Tissue

Wash - In	Off
Wash - Out	Off
TTP	Off
PEI	Off
MIP - time	Off
Measurements	1

Inline - Composing

Inline Composing	Off
Distortion Corr.	Off

Sequence - Part 1

Introduction	On
Dimension	2D
Phase stabilisation	Off
Asymmetric echo	Allowed
Contrasts	1
Flow comp.	No
Multi-slice mode	Sequential
Bandwidth	320 Hz/Px

Sequence - Part 2

Segments	1
----------	---

Sequence - Part 2

Acoustic noise reduction	None
RF pulse type	Fast
Gradient mode	Fast
Excitation	Slice-sel.
RF spoiling	On

Sequence - Assistant

Mode	Off
Allowed delay	0 s

\\USER\CAMRIS\Satterthwaite\SATTERTHWAITE_ABCD_32ch\ABCD_T1w_MPR_vNav_setter

TA: 0.2 s PM: FIX Voxel size: 8.0×8.0×8.0 mmRel. SNR: 1.00 : ABCD

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	Off
Start measurements	Single measurement

Routine

Slab group	1
Slabs	1
Dist. factor	50 %
Position	L0.0 A20.0 H0.0 mm
Orientation	Sagittal
Phase enc. dir.	A >> P
AutoAlign	---
Phase oversampling	0 %
Slice oversampling	0.0 %
Slices per slab	32
FoV read	256 mm
FoV phase	100.0 %
Slice thickness	8.00 mm
TR	9.9 ms
TE	4.6 ms
Averages	1
Concatenations	1
Filter	None
Coil elements	BC

Contrast - Common

TR	9.9 ms
TE	4.6 ms
MTC	Off
Flip angle	2 deg
Fat suppr.	None

Contrast - Dynamic

Averages	1
Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Multiple series	Each measurement

Resolution - Common

FoV read	256 mm
FoV phase	100.0 %
Slice thickness	8.00 mm
Base resolution	32
Phase resolution	100 %
Slice resolution	100 %
Phase partial Fourier	Off
Slice partial Fourier	6/8
Interpolation	Off

Resolution - Filter Image

Distortion Corr.	Off
Prescan Normalize	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off
Hamming	Off

Geometry - Common

Slab group	1
Slabs	1
Dist. factor	50 %
Position	L0.0 A20.0 H0.0 mm
Orientation	Sagittal
Phase enc. dir.	A >> P
Slice oversampling	0.0 %
Slices per slab	32
FoV read	256 mm
FoV phase	100.0 %
Slice thickness	8.00 mm
TR	9.9 ms
Multi-slice mode	Interleaved
Series	Interleaved
Concatenations	1

Geometry - AutoAlign

Slab group	1
Position	L0.0 A20.0 H0.0 mm
Orientation	Sagittal
Phase enc. dir.	A >> P
AutoAlign	---
Initial Position	L0.0 A20.0 H0.0
L	0.0 mm
A	20.0 mm
H	0.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Sagittal

Geometry - Saturation

Fat suppr.	None
Special sat.	None

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	FIX
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Sum of Squares
Save uncombined	Off
Matrix Optimization	Off
AutoAlign	---

System - Miscellaneous

Coil Select Mode	Off - All
------------------	-----------

System - Adjustments

B0 Shim mode	Tune up
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
A >> P	263 mm
R >> L	350 mm
F >> H	350 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Non-sel.

System - Tx/Rx

Frequency 1H	123.255216 MHz
Correction factor	1
Gain	Low
Img. Scale Cor.	10.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	9.9 ms
Concatenations	1

Sequence - Part 1

Introduction	Off
Dimension	3D
Contrasts	1
Multi-slice mode	Interleaved
Free echo spacing	Off
Echo spacing	0.26 ms
Bandwidth	5208 Hz/Px

Sequence - Part 2

EPI factor	32
RF pulse type	Normal
Gradient mode	Fast
Excitation	Non-sel.
RF spoiling	On

Sequence - Special

Protocol filename	MPRAGE
-------------------	--------

\\USER\CAMRIS\Satterthwaite\SATTERTHWAITE_ABCD_32ch\ABCD_T1w_MPR_vNav

TA: 7:12 PM: FIX Voxel size: 1.0×1.0×1.0 mmPAT: 2 Rel. SNR: 1.00 : tfl

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	On
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	On
Start measurements	Single measurement

Routine

Slab group	1
Slabs	1
Dist. factor	50 %
Position	L0.0 A20.0 F30.0 mm
Orientation	Sagittal
Phase enc. dir.	A >> P
AutoAlign	---
Phase oversampling	0 %
Slice oversampling	0.0 %
Slices per slab	176
FoV read	256 mm
FoV phase	100.0 %
Slice thickness	1.00 mm
TR	2500.0 ms
TE	2.9 ms
Averages	1
Concatenations	1
Filter	Prescan Normalize
Coil elements	HEA;HEP

Contrast - Common

TR	2500.0 ms
TE	2.9 ms
Magn. preparation	Non-sel. IR
T1	1070 ms
Flip angle	8.0 deg
Fat suppr.	Water excit. fast
Water suppr.	None

Contrast - Dynamic

Averages	1
Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Multiple series	Each measurement

Resolution - Common

FoV read	256 mm
FoV phase	100.0 %
Slice thickness	1.00 mm
Base resolution	256
Phase resolution	100 %
Slice resolution	100 %
Phase partial Fourier	Off
Slice partial Fourier	Off
Interpolation	Off

Resolution - iPAT

PAT mode	GRAPPA
Accel. factor PE	2
Ref. lines PE	32
Accel. factor 3D	1
Reference scan mode	Integrated

Resolution - Filter Image

Image Filter	Off
Distortion Corr.	Off
Prescan Normalize	On
Unfiltered images	On
Normalize	Off
B1 filter	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off

Geometry - Common

Slab group	1
Slabs	1
Dist. factor	50 %
Position	L0.0 A20.0 F30.0 mm
Orientation	Sagittal
Phase enc. dir.	A >> P
Slice oversampling	0.0 %
Slices per slab	176
FoV read	256 mm
FoV phase	100.0 %
Slice thickness	1.00 mm
TR	2500.0 ms
Multi-slice mode	Single shot
Series	Interleaved
Concatenations	1

Geometry - AutoAlign

Slab group	1
Position	L0.0 A20.0 F30.0 mm
Orientation	Sagittal
Phase enc. dir.	A >> P
AutoAlign	---
Initial Position	L0.0 A20.0 F30.0
R	0.0 mm
A	20.0 mm
F	30.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Sagittal

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	FIX
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L

System - Miscellaneous

Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Adaptive Combine
Save uncombined	Off
Matrix Optimization	Off
AutoAlign	---
Coil Select Mode	Off - All

System - Adjustments

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	L0.0 A20.0 F30.0 mm
Orientation	Sagittal
Rotation	0.00 deg
A >> P	256 mm
F >> H	256 mm
R >> L	176 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Non-sel.

System - Tx/Rx

Frequency 1H	123.255216 MHz
Correction factor	1
Gain	Low
Img. Scale Cor.	5.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	2500.0 ms
Concatenations	1

Physio - Cardiac

Magn. preparation	Non-sel. IR
TI	1070 ms
Fat suppr.	Water excit. fast
Dark blood	Off
FoV read	256 mm
FoV phase	100.0 %
Phase resolution	100 %

Inline - Common

Subtract	Off
Measurements	1
StdDev	Off
Save original images	On

Inline - MIP

MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off

Inline - MIP

Save original images	On
----------------------	----

Inline - Composing

Inline Composing	Off
Distortion Corr.	Off

Sequence - Part 1

Introduction	On
Dimension	3D
Elliptical scanning	Off
Reordering	Linear
Asymmetric echo	Off
Contrasts	1
Flow comp.	No
Multi-slice mode	Single shot
Echo spacing	8.1 ms
Bandwidth	240 Hz/Px

Sequence - Part 2

RF pulse type	Fast
Gradient mode	Fast
Excitation	Non-sel.
RF spoiling	On
Turbo factor	256

Sequence - Special

Readout polarity	Positive
Nav. location	Before
Apply moco to	parent and nav
Remeasure	24 TRs
Reacq. threshold	0.50
Feedback delay	80 ms
Moco ref. image	Use Temp Ref
K-space streaming	None
ABCD navigator	On
Add. grad time	0.0 ms
Apply freq to	parent and nav
Averaging	None

Sequence - Assistant

Mode	Off
------	-----

\\USER\CAMRIS\Satterthwaite\SATTERTHWAITE_ABCD_32ch\ABCD_T2w_SPC_vNav_setter

TA: 0.3 s PM: FIX Voxel size: 8.0×8.0×8.0 mmRel. SNR: 1.00 : ABCD

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	On
Wait for user to start	On
Start measurements	Single measurement

Routine

Slab group	1
Slabs	1
Dist. factor	50 %
Position	L0.0 A20.0 H0.0 mm
Orientation	Sagittal
Phase enc. dir.	A >> P
AutoAlign	---
Phase oversampling	0 %
Slice oversampling	0.0 %
Slices per slab	32
FoV read	256 mm
FoV phase	100.0 %
Slice thickness	8.00 mm
TR	13.0 ms
TE	6.0 ms
Averages	1
Concatenations	1
Filter	None
Coil elements	BC

Contrast - Common

TR	13.0 ms
TE	6.0 ms
MTC	Off
Flip angle	2 deg
Fat suppr.	Water excit. normal

Contrast - Dynamic

Averages	1
Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Multiple series	Each measurement

Resolution - Common

FoV read	256 mm
FoV phase	100.0 %
Slice thickness	8.00 mm
Base resolution	32
Phase resolution	100 %
Slice resolution	100 %
Phase partial Fourier	Off
Slice partial Fourier	6/8
Interpolation	Off

Resolution - Filter Image

Distortion Corr.	Off
Prescan Normalize	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off
Hamming	Off

Geometry - Common

Slab group	1
Slabs	1
Dist. factor	50 %
Position	L0.0 A20.0 H0.0 mm
Orientation	Sagittal
Phase enc. dir.	A >> P
Slice oversampling	0.0 %
Slices per slab	32
FoV read	256 mm
FoV phase	100.0 %
Slice thickness	8.00 mm
TR	13.0 ms
Multi-slice mode	Interleaved
Series	Interleaved
Concatenations	1

Geometry - AutoAlign

Slab group	1
Position	L0.0 A20.0 H0.0 mm
Orientation	Sagittal
Phase enc. dir.	A >> P
AutoAlign	---
Initial Position	L0.0 A20.0 H0.0
L	0.0 mm
A	20.0 mm
H	0.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Sagittal

Geometry - Saturation

Fat suppr.	Water excit. normal
Special sat.	None

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	FIX
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Sum of Squares
Save uncombined	Off
Matrix Optimization	Off
AutoAlign	---

System - Miscellaneous

Coil Select Mode	Off - All
------------------	-----------

System - Adjustments

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	L0.0 A20.0 H0.0 mm
Orientation	Sagittal
Rotation	0.00 deg
A >> P	256 mm
F >> H	256 mm
R >> L	256 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Non-sel.

System - Tx/Rx

Frequency 1H	123.255216 MHz
Correction factor	1
Gain	Low
Img. Scale Cor.	10.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	13.0 ms
Concatenations	1

Sequence - Part 1

Introduction	Off
Dimension	3D
Contrasts	1
Multi-slice mode	Interleaved
Free echo spacing	Off
Echo spacing	0.26 ms
Bandwidth	5208 Hz/Px

Sequence - Part 2

EPI factor	32
RF pulse type	Normal
Gradient mode	Fast
Excitation	Non-sel.
RF spoiling	On

Sequence - Special

Protocol filename	T2-SPACE
-------------------	----------

\\USER\CAMRIS\Satterthwaite\SATTERTHWAITE_ABCD_32ch\ABCD_T2w_SPC_vNav

TA: 6:35 PM: FIX Voxel size: 1.0×1.0×1.0 mmPAT: 2 Rel. SNR: 1.00 : spc

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	On
Wait for user to start	Off
Start measurements	Single measurement

Routine

Slab group	1
Slabs	1
Position	L0.0 A20.0 F30.0 mm
Orientation	Sagittal
Phase enc. dir.	A >> P
AutoAlign	---
Phase oversampling	0 %
Slice oversampling	0.0 %
Slices per slab	176
FoV read	256 mm
FoV phase	100.0 %
Slice thickness	1.00 mm
TR	3200 ms
TE	565 ms
Averages	1.0
Concatenations	1
Filter	Prescan Normalize
Coil elements	HEA;HEP

Contrast - Common

TR	3200 ms
TE	565 ms
MTC	Off
Magn. preparation	None
Fat suppr.	None
Blood suppr.	Off
Restore magn.	Off

Contrast - Dynamic

Averages	1.0
Reconstruction	Magnitude
Measurements	1
Multiple series	Each measurement

Resolution - Common

FoV read	256 mm
FoV phase	100.0 %
Slice thickness	1.00 mm
Base resolution	256
Phase resolution	100 %
Slice resolution	100 %
Phase partial Fourier	Allowed
Slice partial Fourier	Off
Interpolation	Off

Resolution - iPAT

PAT mode	GRAPPA
Accel. factor PE	2
Ref. lines PE	32
Accel. factor 3D	1
Reference scan mode	Integrated

Resolution - Filter Image

Image Filter	Off
Distortion Corr.	Off
Prescan Normalize	On
Unfiltered images	On
Normalize	Off
B1 filter	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off

Geometry - Common

Slab group	1
Slabs	1
Position	L0.0 A20.0 F30.0 mm
Orientation	Sagittal
Phase enc. dir.	A >> P
Slice oversampling	0.0 %
Slices per slab	176
FoV read	256 mm
FoV phase	100.0 %
Slice thickness	1.00 mm
TR	3200 ms
Concatenations	1

Geometry - AutoAlign

Slab group	1
Position	L0.0 A20.0 F30.0 mm
Orientation	Sagittal
Phase enc. dir.	A >> P
AutoAlign	---
Initial Position	L0.0 A20.0 F30.0
R	0.0 mm
A	20.0 mm
F	30.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Sagittal

Geometry - Saturation

Fat suppr.	None
Restore magn.	Off
Special sat.	None

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	FIX
Table position	H
Table position	0 mm

System - Miscellaneous

MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Adaptive Combine
Save uncombined	Off
Matrix Optimization	Off
AutoAlign	---
Coil Select Mode	Off - All

System - Adjustments

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	L0.0 A20.0 F30.0 mm
Orientation	Sagittal
Rotation	0.00 deg
A >> P	256 mm
F >> H	256 mm
R >> L	176 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Non-sel.

System - Tx/Rx

Frequency 1H	123.255216 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	5.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
Trigger delay	0 ms
TR	3200 ms
Concatenations	1

Physio - Cardiac

Magn. preparation	None
Fat suppr.	None
Dark blood	Off
FoV read	256 mm
FoV phase	100.0 %
Phase resolution	100 %

Inline - Common

Subtract	Off
Measurements	1
StdDev	Off
Save original images	On

Inline - MIP

MIP-Sag	Off
MIP-Cor	Off

Inline - MIP

MIP-Tra	Off
MIP-Time	Off
Save original images	On

Inline - Composing

Inline Composing	Off
Distortion Corr.	Off

Sequence - Part 1

Introduction	On
Dimension	3D
Elliptical scanning	On
Reordering	Linear
Flow comp.	No
Echo spacing	6.28 ms
Adiabatic-mode	Off
Bandwidth	241 Hz/Px

Sequence - Part 2

Echo train duration	1187 ms
RF pulse type	Fast
Gradient mode	Fast
Excitation	Non-sel.
Flip angle mode	T2 var
Turbo factor	200

Sequence - Special

Include nav.	On
Apply moco to	parent and nav
Remeasure	18 TRs
Reacq. threshold	0.50
Feedback delay	80 ms
Moco ref. image	Use Temp Ref
K-space streaming	None
ABCD navigator	On
Apply freq to	parent and nav

Sequence - Assistant

Allowed delay	0 s
---------------	-----

\\USER\CAMRIS\Satterthwaite\SATTERTHWAITE_ABCD_32ch\ABCD_fmRI_DistortionMap_PA

TA: 0:14 PM: FIX Voxel size: 2.4x2.4x2.4 mmPAT: Off Rel. SNR: 1.00 : epse

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	Off
Start measurements	Single measurement

Routine

Slice group	1
Slices	60
Dist. factor	0 %
Position	L0.0 A10.0 H10.0 mm
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	---
Phase oversampling	0 %
FoV read	216 mm
FoV phase	100.0 %
Slice thickness	2.4 mm
TR	7030 ms
TE	80.0 ms
Averages	1
Concatenations	1
Filter	None
Coil elements	HEA;HEP

Contrast - Common

TR	7030 ms
TE	80.0 ms
MTC	Off
Magn. preparation	None
Fat suppr.	Fat sat.
Fat sat. mode	Weak

Contrast - Dynamic

Averages	1
Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Delay in TR	0 ms
Multiple series	Off

Resolution - Common

FoV read	216 mm
FoV phase	100.0 %
Slice thickness	2.4 mm
Base resolution	90
Phase resolution	100 %
Phase partial Fourier	Off
Interpolation	Off

Resolution - iPAT

Accel. mode	None
-------------	------

Resolution - Filter Image

Distortion Corr.	Off
Prescan Normalize	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off
Hamming	Off

Geometry - Common

Slice group	1
Slices	60
Dist. factor	0 %
Position	L0.0 A10.0 H10.0 mm
Orientation	Transversal
Phase enc. dir.	A >> P
FoV read	216 mm
FoV phase	100.0 %
Slice thickness	2.4 mm
TR	7030 ms
Multi-slice mode	Interleaved
Series	Interleaved
Concatenations	1

Geometry - AutoAlign

Slice group	1
Position	L0.0 A10.0 H10.0 mm
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	---
Initial Position	L0.0 A10.0 H10.0
L	0.0 mm
A	10.0 mm
H	10.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Transversal

Geometry - Saturation

Fat suppr.	Fat sat.
Fat sat. mode	Weak
Special sat.	None

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	FIX
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Adaptive Combine
Save uncombined	Off
Matrix Optimization	Off
AutoAlign	---
Coil Select Mode	Off - All

System - Adjustments

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

Sequence - Special

Dummy Scans	1
Reverse Phase Encoding	On

System - Adjust Volume

Position	L0.0 A10.0 H10.0 mm
Orientation	Transversal
Rotation	0.00 deg
A >> P	216 mm
R >> L	216 mm
F >> H	144 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
--------------	----------

System - Tx/Rx

Frequency 1H	123.255216 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	2.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	7030 ms
Concatenations	1

Inline - Common

Subtract	Off
Measurements	1
StdDev	Off
Save original images	On

Inline - MIP

MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

Inline - Composing

Inline Composing	Off
Distortion Corr.	Off

Sequence - Part 1

Introduction	Off
Multi-slice mode	Interleaved
Free echo spacing	On
Echo spacing	0.51 ms
Bandwidth	2778 Hz/Px

Sequence - Part 2

EPI factor	90
RF pulse type	Normal
Gradient mode	Performance

\\USER\CAMRIS\Satterthwaite\SATTERTHWAITE_ABCD_32ch\ABCD_fmri_DistortionMap_AP

TA: 0:14 PM: FIX Voxel size: 2.4x2.4x2.4 mmPAT: Off Rel. SNR: 1.00 : epse

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	On
Wait for user to start	Off
Start measurements	Single measurement

Routine

Slice group	1
Slices	60
Dist. factor	0 %
Position	L0.0 A10.0 H10.0 mm
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	---
Phase oversampling	0 %
FoV read	216 mm
FoV phase	100.0 %
Slice thickness	2.4 mm
TR	7030 ms
TE	80.0 ms
Averages	1
Concatenations	1
Filter	None
Coil elements	HEA;HEP

Contrast - Common

TR	7030 ms
TE	80.0 ms
MTC	Off
Magn. preparation	None
Fat suppr.	Fat sat.
Fat sat. mode	Weak

Contrast - Dynamic

Averages	1
Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Delay in TR	0 ms
Multiple series	Off

Resolution - Common

FoV read	216 mm
FoV phase	100.0 %
Slice thickness	2.4 mm
Base resolution	90
Phase resolution	100 %
Phase partial Fourier	Off
Interpolation	Off

Resolution - iPAT

Accel. mode	None
-------------	------

Resolution - Filter Image

Distortion Corr.	Off
Prescan Normalize	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off
Hamming	Off

Geometry - Common

Slice group	1
Slices	60
Dist. factor	0 %
Position	L0.0 A10.0 H10.0 mm
Orientation	Transversal
Phase enc. dir.	A >> P
FoV read	216 mm
FoV phase	100.0 %
Slice thickness	2.4 mm
TR	7030 ms
Multi-slice mode	Interleaved
Series	Interleaved
Concatenations	1

Geometry - AutoAlign

Slice group	1
Position	L0.0 A10.0 H10.0 mm
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	---
Initial Position	L0.0 A10.0 H10.0
L	0.0 mm
A	10.0 mm
H	10.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Transversal

Geometry - Saturation

Fat suppr.	Fat sat.
Fat sat. mode	Weak
Special sat.	None

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	FIX
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Adaptive Combine
Save uncombined	Off
Matrix Optimization	Off
AutoAlign	---
Coil Select Mode	Off - All

System - Adjustments

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

Sequence - Special

Dummy Scans	1
Reverse Phase Encoding	Off

System - Adjust Volume

Position	L0.0 A10.0 H10.0 mm
Orientation	Transversal
Rotation	0.00 deg
A >> P	216 mm
R >> L	216 mm
F >> H	144 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
--------------	----------

System - Tx/Rx

Frequency 1H	123.255216 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	2.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	7030 ms
Concatenations	1

Inline - Common

Subtract	Off
Measurements	1
StdDev	Off
Save original images	On

Inline - MIP

MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

Inline - Composing

Inline Composing	Off
Distortion Corr.	Off

Sequence - Part 1

Introduction	Off
Multi-slice mode	Interleaved
Free echo spacing	On
Echo spacing	0.51 ms
Bandwidth	2778 Hz/Px

Sequence - Part 2

EPI factor	90
RF pulse type	Normal
Gradient mode	Performance

\\USER\CAMRIS\Satterthwaite\SATTERTHWAITE_ABCD_32ch\ABCD_fmri_rest

TA: 5:11 PM: FIX Voxel size: 2.4x2.4x2.4 mmPAT: 6 Rel. SNR: 1.00 : epfid

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	On
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	On
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	On
Wait for user to start	On
Start measurements	Single measurement

Routine

Slice group	1
Slices	60
Dist. factor	0 %
Position	L0.0 A10.0 H10.0 mm
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	---
Phase oversampling	0 %
FoV read	216 mm
FoV phase	100.0 %
Slice thickness	2.4 mm
TR	800 ms
TE	30.0 ms
Averages	1
Concatenations	1
Filter	None
Coil elements	HEA;HEP

Contrast - Common

TR	800 ms
TE	30.0 ms
MTC	Off
Flip angle	52 deg
Fat suppr.	Fat sat.

Contrast - Dynamic

Averages	1
Averaging mode	Long term
Reconstruction	Magnitude
Measurements	383
Delay in TR	0 ms
Multiple series	Off

Resolution - Common

FoV read	216 mm
FoV phase	100.0 %
Slice thickness	2.4 mm
Base resolution	90
Phase resolution	100 %
Phase partial Fourier	Off
Interpolation	Off

Resolution - iPAT

Accel. mode	Slice accel.
Accel. factor PE	1
Ref. lines PE	22

Resolution - iPAT

Accel. factor slice	6
Reference scan mode	EPI/separate

Resolution - Filter Image

Distortion Corr.	Off
Prescan Normalize	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off
Hamming	Off

Geometry - Common

Slice group	1
Slices	60
Dist. factor	0 %
Position	L0.0 A10.0 H10.0 mm
Orientation	Transversal
Phase enc. dir.	A >> P
FoV read	216 mm
FoV phase	100.0 %
Slice thickness	2.4 mm
TR	800 ms
Multi-slice mode	Interleaved
Series	Interleaved
Concatenations	1

Geometry - AutoAlign

Slice group	1
Position	L0.0 A10.0 H10.0 mm
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	---
Initial Position	L0.0 A10.0 H10.0
L	0.0 mm
A	10.0 mm
H	10.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Transversal

Geometry - Saturation

Fat suppr.	Fat sat.
Special sat.	None

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	FIX
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Sum of Squares
Matrix Optimization	Off

System - Miscellaneous

AutoAlign	---
Coil Select Mode	Off - All

System - Adjustments

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	L0.0 A10.0 H10.0 mm
Orientation	Transversal
Rotation	0.00 deg
A >> P	216 mm
R >> L	216 mm
F >> H	144 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Standard

System - Tx/Rx

Frequency 1H	123.255216 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	Pulse/Trigger
Average cycle	899 ± 3 ms
Average cycle	911 ± 3 ms
Acquisition window	833 ms
Trigger pulse	1
Trigger delay	0 ms
TR	800 ms
Concatenations	1
Phases	1

BOLD

GLM Statistics	Off
Dynamic t-maps	Off
Ignore meas. at start	0
Ignore after transition	0
Model transition states	Off
Temp. highpass filter	Off
Threshold	4.00
Paradigm size	3
Meas[1]	Baseline
Meas[2]	Baseline
Meas[3]	Active
Motion correction	Off
Spatial filter	Off
Measurements	383
Delay in TR	0 ms
Multiple series	Off

Sequence - Part 1

Introduction	Off
Multi-slice mode	Interleaved
Free echo spacing	Off
Echo spacing	0.51 ms
Bandwidth	2778 Hz/Px

Sequence - Part 2

EPI factor	90
RF pulse type	Normal
Gradient mode	Fast
Excitation	Standard

Sequence - pTX Pulses**Sequence - Special**

Dummy Scans	0
K-space streaming	None
Physio recording	Continuous
Reverse Phase Encoding	On

\\USER\CAMRIS\Satterthwaite\SATTERTHWAITE_ABCD_32ch\ABCD_fmri_rest

TA: 5:11 PM: FIX Voxel size: 2.4x2.4x2.4 mmPAT: 6 Rel. SNR: 1.00 : epfid

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	On
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	On
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	On
Wait for user to start	On
Start measurements	Single measurement

Routine

Slice group	1
Slices	60
Dist. factor	0 %
Position	L0.0 A10.0 H10.0 mm
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	---
Phase oversampling	0 %
FoV read	216 mm
FoV phase	100.0 %
Slice thickness	2.4 mm
TR	800 ms
TE	30.0 ms
Averages	1
Concatenations	1
Filter	None
Coil elements	HEA;HEP

Contrast - Common

TR	800 ms
TE	30.0 ms
MTC	Off
Flip angle	52 deg
Fat suppr.	Fat sat.

Contrast - Dynamic

Averages	1
Averaging mode	Long term
Reconstruction	Magnitude
Measurements	383
Delay in TR	0 ms
Multiple series	Off

Resolution - Common

FoV read	216 mm
FoV phase	100.0 %
Slice thickness	2.4 mm
Base resolution	90
Phase resolution	100 %
Phase partial Fourier	Off
Interpolation	Off

Resolution - iPAT

Accel. mode	Slice accel.
Accel. factor PE	1
Ref. lines PE	22

Resolution - iPAT

Accel. factor slice	6
Reference scan mode	EPI/separate

Resolution - Filter Image

Distortion Corr.	Off
Prescan Normalize	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off
Hamming	Off

Geometry - Common

Slice group	1
Slices	60
Dist. factor	0 %
Position	L0.0 A10.0 H10.0 mm
Orientation	Transversal
Phase enc. dir.	A >> P
FoV read	216 mm
FoV phase	100.0 %
Slice thickness	2.4 mm
TR	800 ms
Multi-slice mode	Interleaved
Series	Interleaved
Concatenations	1

Geometry - AutoAlign

Slice group	1
Position	L0.0 A10.0 H10.0 mm
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	---
Initial Position	L0.0 A10.0 H10.0
L	0.0 mm
A	10.0 mm
H	10.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Transversal

Geometry - Saturation

Fat suppr.	Fat sat.
Special sat.	None

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	FIX
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Sum of Squares
Matrix Optimization	Off

System - Miscellaneous

AutoAlign	---
Coil Select Mode	Off - All

System - Adjustments

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	L0.0 A10.0 H10.0 mm
Orientation	Transversal
Rotation	0.00 deg
A >> P	216 mm
R >> L	216 mm
F >> H	144 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Standard

System - Tx/Rx

Frequency 1H	123.255216 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	Pulse/Trigger
Average cycle	900 ± 4 ms
Average cycle	918 ± 4 ms
Acquisition window	833 ms
Trigger pulse	1
Trigger delay	0 ms
TR	800 ms
Concatenations	1
Phases	1

BOLD

GLM Statistics	Off
Dynamic t-maps	Off
Ignore meas. at start	0
Ignore after transition	0
Model transition states	Off
Temp. highpass filter	Off
Threshold	4.00
Paradigm size	3
Meas[1]	Baseline
Meas[2]	Baseline
Meas[3]	Active
Motion correction	Off
Spatial filter	Off
Measurements	383
Delay in TR	0 ms
Multiple series	Off

Sequence - Part 1

Introduction	Off
Multi-slice mode	Interleaved
Free echo spacing	Off
Echo spacing	0.51 ms
Bandwidth	2778 Hz/Px

Sequence - Part 2

EPI factor	90
RF pulse type	Normal
Gradient mode	Fast
Excitation	Standard

Sequence - pTX Pulses**Sequence - Special**

Dummy Scans	0
K-space streaming	None
Physio recording	Continuous
Reverse Phase Encoding	On

\\USER\CAMRIS\Satterthwaite\SATTERTHWAITE_ABCD_32ch\ABCD_dMRI_DistortionMap_PA

TA: 0:25 PM: REF Voxel size: 1.7×1.7×1.7 mmPAT: Off Rel. SNR: 1.00 : epse

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	On
Start measurements	Single measurement

Routine

Slice group	1
Slices	81
Dist. factor	0 %
Position	L0.0 A10.0 H10.0 mm
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	---
Phase oversampling	0 %
FoV read	240 mm
FoV phase	100.0 %
Slice thickness	1.7 mm
TR	12400 ms
TE	89.0 ms
Averages	1
Concatenations	1
Filter	None
Coil elements	HEA;HEP

Contrast - Common

TR	12400 ms
TE	89.0 ms
MTC	Off
Magn. preparation	None
Fat suppr.	Fat sat.
Fat sat. mode	Strong

Contrast - Dynamic

Averages	1
Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Delay in TR	0 ms
Multiple series	Off

Resolution - Common

FoV read	240 mm
FoV phase	100.0 %
Slice thickness	1.7 mm
Base resolution	140
Phase resolution	100 %
Phase partial Fourier	6/8
Interpolation	Off

Resolution - iPAT

Accel. mode	None
-------------	------

Resolution - Filter Image

Distortion Corr.	Off
Prescan Normalize	Off
Dynamic Field Corr.	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off

Geometry - Common

Slice group	1
Slices	81
Dist. factor	0 %
Position	L0.0 A10.0 H10.0 mm
Orientation	Transversal
Phase enc. dir.	A >> P
FoV read	240 mm
FoV phase	100.0 %
Slice thickness	1.7 mm
TR	12400 ms
Multi-slice mode	Interleaved
Series	Interleaved
Concatenations	1

Geometry - AutoAlign

Slice group	1
Position	L0.0 A10.0 H10.0 mm
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	---
Initial Position	L0.0 A10.0 H10.0
L	0.0 mm
A	10.0 mm
H	10.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Transversal

Geometry - Saturation

Fat suppr.	Fat sat.
Fat sat. mode	Strong
Special sat.	None

Geometry - Navigator**Geometry - Tim Planning Suite**

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	REF
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Adaptive Combine
Matrix Optimization	Off
AutoAlign	---

System - Miscellaneous

Coil Select Mode	Off - All
------------------	-----------

System - Adjustments

B0 Shim mode	Advanced
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	L0.0 A10.0 H10.0 mm
Orientation	Transversal
Rotation	0.00 deg
A >> P	240 mm
R >> L	240 mm
F >> H	138 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Standard

System - Tx/Rx

Frequency 1H	123.255216 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	2.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	12400 ms
Concatenations	1

Physio - PACE

Resp. control	Off
Concatenations	1

Diff - Neuro

Diffusion mode	Free
Diff. directions	102
Diffusion Scheme	Monopolar
Diff. weightings	1
b-value	0 s/mm ²
b-value	1
Diff. weighted images	On
Trace weighted images	Off
ADC maps	Off
FA maps	Off
Mosaic	Off
Tensor	Off
Noise level	40

Diff - Body

Diffusion mode	Free
Diff. directions	102
Diffusion Scheme	Monopolar
Diff. weightings	1
b-value	0 s/mm ²
b-value	1

Diff - Body

Diff. weighted images	On
Trace weighted images	Off
ADC maps	Off
Exponential ADC Maps	Off
FA maps	Off
Invert Gray Scale	Off
Calculated Image	Off
b-Value >=	0 s/mm ²
Noise level	40

Diff - Composing

Inline Composing	Off
Distortion Corr.	Off

Sequence - Part 1

Introduction	Off
Optimization	None
Multi-slice mode	Interleaved
Free echo spacing	On
Echo spacing	0.69 ms
Bandwidth	1700 Hz/Px

Sequence - Part 2

EPI factor	140
RF pulse type	Normal
Gradient mode	Performance
Excitation	Standard

Sequence - pTX Pulses

Sequence - Special

K-space streaming	None
Reverse Phase Encoding	On

\\USER\CAMRIS\Satterthwaite\SATTERTHWAITE_ABCD_32ch\ABCD_dMRI_DistortionMap_AP

TA: 0:25 PM: FIX Voxel size: 1.7×1.7×1.7 mmPAT: Off Rel. SNR: 1.00 : epse

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	On
Wait for user to start	Off
Start measurements	Single measurement

Routine

Slice group	1
Slices	81
Dist. factor	0 %
Position	L0.0 A10.0 H10.0 mm
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	---
Phase oversampling	0 %
FoV read	240 mm
FoV phase	100.0 %
Slice thickness	1.7 mm
TR	12400 ms
TE	89.0 ms
Averages	1
Concatenations	1
Filter	None
Coil elements	HEA;HEP

Contrast - Common

TR	12400 ms
TE	89.0 ms
MTC	Off
Magn. preparation	None
Fat suppr.	Fat sat.
Fat sat. mode	Strong

Contrast - Dynamic

Averages	1
Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Delay in TR	0 ms
Multiple series	Off

Resolution - Common

FoV read	240 mm
FoV phase	100.0 %
Slice thickness	1.7 mm
Base resolution	140
Phase resolution	100 %
Phase partial Fourier	6/8
Interpolation	Off

Resolution - iPAT

Accel. mode	None
-------------	------

Resolution - Filter Image

Distortion Corr.	Off
Prescan Normalize	Off
Dynamic Field Corr.	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off

Geometry - Common

Slice group	1
Slices	81
Dist. factor	0 %
Position	L0.0 A10.0 H10.0 mm
Orientation	Transversal
Phase enc. dir.	A >> P
FoV read	240 mm
FoV phase	100.0 %
Slice thickness	1.7 mm
TR	12400 ms
Multi-slice mode	Interleaved
Series	Interleaved
Concatenations	1

Geometry - AutoAlign

Slice group	1
Position	L0.0 A10.0 H10.0 mm
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	---
Initial Position	L0.0 A10.0 H10.0
L	0.0 mm
A	10.0 mm
H	10.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Transversal

Geometry - Saturation

Fat suppr.	Fat sat.
Fat sat. mode	Strong
Special sat.	None

Geometry - Navigator**Geometry - Tim Planning Suite**

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	FIX
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Adaptive Combine
Matrix Optimization	Off
AutoAlign	---

System - Miscellaneous

Coil Select Mode	Off - All
------------------	-----------

System - Adjustments

B0 Shim mode	Advanced
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	L0.0 A10.0 H10.0 mm
Orientation	Transversal
Rotation	0.00 deg
A >> P	240 mm
R >> L	240 mm
F >> H	138 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Standard

System - Tx/Rx

Frequency 1H	123.255216 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	2.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	12400 ms
Concatenations	1

Physio - PACE

Resp. control	Off
Concatenations	1

Diff - Neuro

Diffusion mode	Free
Diff. directions	102
Diffusion Scheme	Monopolar
Diff. weightings	1
b-value	0 s/mm ²
b-value	1
Diff. weighted images	On
Trace weighted images	Off
ADC maps	Off
FA maps	Off
Mosaic	Off
Tensor	Off
Noise level	40

Diff - Body

Diffusion mode	Free
Diff. directions	102
Diffusion Scheme	Monopolar
Diff. weightings	1
b-value	0 s/mm ²
b-value	1

Diff - Body

Diff. weighted images	On
Trace weighted images	Off
ADC maps	Off
Exponential ADC Maps	Off
FA maps	Off
Invert Gray Scale	Off
Calculated Image	Off
b-Value >=	0 s/mm ²
Noise level	40

Diff - Composing

Inline Composing	Off
Distortion Corr.	Off

Sequence - Part 1

Introduction	Off
Optimization	None
Multi-slice mode	Interleaved
Free echo spacing	On
Echo spacing	0.69 ms
Bandwidth	1700 Hz/Px

Sequence - Part 2

EPI factor	140
RF pulse type	Normal
Gradient mode	Performance
Excitation	Standard

Sequence - pTX Pulses**Sequence - Special**

K-space streaming	None
Reverse Phase Encoding	Off

\\USER\CAMRIS\Satterthwaite\SATTERTHWAITE_ABCD_32ch\ABCD_dMRI

TA: 7:29 PM: FIX Voxel size: 1.7×1.7×1.7 mmPAT: 3 Rel. SNR: 1.00 : epse

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	On
Wait for user to start	On
Start measurements	Single measurement

Routine

Slice group	1
Slices	81
Dist. factor	0 %
Position	L0.0 A10.0 H10.0 mm
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	---
Phase oversampling	0 %
FoV read	240 mm
FoV phase	100.0 %
Slice thickness	1.7 mm
TR	4200 ms
TE	89.0 ms
Averages	1
Concatenations	1
Filter	None
Coil elements	HEA;HEP

Contrast - Common

TR	4200 ms
TE	89.0 ms
MTC	Off
Magn. preparation	None
Fat suppr.	Fat sat.
Fat sat. mode	Strong

Contrast - Dynamic

Averages	1
Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Delay in TR	0 ms
Multiple series	Off

Resolution - Common

FoV read	240 mm
FoV phase	100.0 %
Slice thickness	1.7 mm
Base resolution	140
Phase resolution	100 %
Phase partial Fourier	6/8
Interpolation	Off

Resolution - iPAT

Accel. mode	Slice accel.
Accel. factor PE	1

Resolution - iPAT

Ref. lines PE	71
Accel. factor slice	3
Reference scan mode	EPI/separate

Resolution - Filter Image

Distortion Corr.	Off
Prescan Normalize	Off
Dynamic Field Corr.	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off

Geometry - Common

Slice group	1
Slices	81
Dist. factor	0 %
Position	L0.0 A10.0 H10.0 mm
Orientation	Transversal
Phase enc. dir.	A >> P
FoV read	240 mm
FoV phase	100.0 %
Slice thickness	1.7 mm
TR	4200 ms
Multi-slice mode	Interleaved
Series	Interleaved
Concatenations	1

Geometry - AutoAlign

Slice group	1
Position	L0.0 A10.0 H10.0 mm
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	---
Initial Position	L0.0 A10.0 H10.0
L	0.0 mm
A	10.0 mm
H	10.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Transversal

Geometry - Saturation

Fat suppr.	Fat sat.
Fat sat. mode	Strong
Special sat.	None

Geometry - Navigator**Geometry - Tim Planning Suite**

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	FIX
Table position	H
Table position	0 mm
MSMA	S - C - T

System - Miscellaneous

Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Adaptive Combine
Matrix Optimization	Off
AutoAlign	---
Coil Select Mode	Off - All

System - Adjustments

B0 Shim mode	Advanced
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	L0.0 A10.0 H10.0 mm
Orientation	Transversal
Rotation	0.00 deg
A >> P	240 mm
R >> L	240 mm
F >> H	138 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Standard

System - Tx/Rx

Frequency 1H	123.255216 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	3.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	4200 ms
Concatenations	1

Physio - PACE

Resp. control	Off
Concatenations	1

Diff - Neuro

Diffusion mode	Free
Diff. directions	102
Diffusion Scheme	Monopolar
Diff. weightings	2
b-value 1	0 s/mm ²
b-value 2	3000 s/mm ²
b-value 1	1
b-value 2	1
Diff. weighted images	On
Trace weighted images	Off
ADC maps	Off
FA maps	Off
Mosaic	On
Tensor	Off
Noise level	40

Diff - Body

Diffusion mode	Free
Diff. directions	102
Diffusion Scheme	Monopolar
Diff. weightings	2
b-value 1	0 s/mm ²
b-value 2	3000 s/mm ²
b-value 1	1
b-value 2	1
Diff. weighted images	On
Trace weighted images	Off
ADC maps	Off
Exponential ADC Maps	Off
FA maps	Off
Invert Gray Scale	Off
Calculated Image	Off
b-Value >=	0 s/mm ²
Noise level	40

Diff - Composing

Inline Composing	Off
Distortion Corr.	Off

Sequence - Part 1

Introduction	Off
Optimization	None
Multi-slice mode	Interleaved
Free echo spacing	On
Echo spacing	0.69 ms
Bandwidth	1700 Hz/Px

Sequence - Part 2

EPI factor	140
RF pulse type	Normal
Gradient mode	Performance
Excitation	Standard

Sequence - pTX Pulses**Sequence - Special**

K-space streaming	None
Reverse Phase Encoding	On

\\USER\CAMRIS\Satterthwaite\SATTERTHWAITE_ABCD_32ch\ABCD_fmri_DistortionMap_PA

TA: 0:14 PM: FIX Voxel size: 2.4x2.4x2.4 mmPAT: Off Rel. SNR: 1.00 : epse

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	Off
Start measurements	Single measurement

Routine

Slice group	1
Slices	60
Dist. factor	0 %
Position	L0.0 A10.0 H10.0 mm
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	---
Phase oversampling	0 %
FoV read	216 mm
FoV phase	100.0 %
Slice thickness	2.4 mm
TR	7030 ms
TE	80.0 ms
Averages	1
Concatenations	1
Filter	None
Coil elements	HEA;HEP

Contrast - Common

TR	7030 ms
TE	80.0 ms
MTC	Off
Magn. preparation	None
Fat suppr.	Fat sat.
Fat sat. mode	Weak

Contrast - Dynamic

Averages	1
Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Delay in TR	0 ms
Multiple series	Off

Resolution - Common

FoV read	216 mm
FoV phase	100.0 %
Slice thickness	2.4 mm
Base resolution	90
Phase resolution	100 %
Phase partial Fourier	Off
Interpolation	Off

Resolution - iPAT

Accel. mode	None
-------------	------

Resolution - Filter Image

Distortion Corr.	Off
Prescan Normalize	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off
Hamming	Off

Geometry - Common

Slice group	1
Slices	60
Dist. factor	0 %
Position	L0.0 A10.0 H10.0 mm
Orientation	Transversal
Phase enc. dir.	A >> P
FoV read	216 mm
FoV phase	100.0 %
Slice thickness	2.4 mm
TR	7030 ms
Multi-slice mode	Interleaved
Series	Interleaved
Concatenations	1

Geometry - AutoAlign

Slice group	1
Position	L0.0 A10.0 H10.0 mm
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	---
Initial Position	L0.0 A10.0 H10.0
L	0.0 mm
A	10.0 mm
H	10.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Transversal

Geometry - Saturation

Fat suppr.	Fat sat.
Fat sat. mode	Weak
Special sat.	None

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	FIX
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Adaptive Combine
Save uncombined	Off
Matrix Optimization	Off
AutoAlign	---
Coil Select Mode	Off - All

System - Adjustments

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

Sequence - Special

Dummy Scans	1
Reverse Phase Encoding	On

System - Adjust Volume

Position	L0.0 A10.0 H10.0 mm
Orientation	Transversal
Rotation	0.00 deg
A >> P	216 mm
R >> L	216 mm
F >> H	144 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
--------------	----------

System - Tx/Rx

Frequency 1H	123.255216 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	2.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	7030 ms
Concatenations	1

Inline - Common

Subtract	Off
Measurements	1
StdDev	Off
Save original images	On

Inline - MIP

MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

Inline - Composing

Inline Composing	Off
Distortion Corr.	Off

Sequence - Part 1

Introduction	Off
Multi-slice mode	Interleaved
Free echo spacing	On
Echo spacing	0.51 ms
Bandwidth	2778 Hz/Px

Sequence - Part 2

EPI factor	90
RF pulse type	Normal
Gradient mode	Performance

\\USER\CAMRIS\Satterthwaite\SATTERTHWAITE_ABCD_32ch\ABCD_fmri_DistortionMap_AP

TA: 0:14 PM: FIX Voxel size: 2.4x2.4x2.4 mmPAT: Off Rel. SNR: 1.00 : epse

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	On
Wait for user to start	Off
Start measurements	Single measurement

Routine

Slice group	1
Slices	60
Dist. factor	0 %
Position	L0.0 A10.0 H10.0 mm
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	---
Phase oversampling	0 %
FoV read	216 mm
FoV phase	100.0 %
Slice thickness	2.4 mm
TR	7030 ms
TE	80.0 ms
Averages	1
Concatenations	1
Filter	None
Coil elements	HEA;HEP

Contrast - Common

TR	7030 ms
TE	80.0 ms
MTC	Off
Magn. preparation	None
Fat suppr.	Fat sat.
Fat sat. mode	Weak

Contrast - Dynamic

Averages	1
Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Delay in TR	0 ms
Multiple series	Off

Resolution - Common

FoV read	216 mm
FoV phase	100.0 %
Slice thickness	2.4 mm
Base resolution	90
Phase resolution	100 %
Phase partial Fourier	Off
Interpolation	Off

Resolution - iPAT

Accel. mode	None
-------------	------

Resolution - Filter Image

Distortion Corr.	Off
Prescan Normalize	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off
Hamming	Off

Geometry - Common

Slice group	1
Slices	60
Dist. factor	0 %
Position	L0.0 A10.0 H10.0 mm
Orientation	Transversal
Phase enc. dir.	A >> P
FoV read	216 mm
FoV phase	100.0 %
Slice thickness	2.4 mm
TR	7030 ms
Multi-slice mode	Interleaved
Series	Interleaved
Concatenations	1

Geometry - AutoAlign

Slice group	1
Position	L0.0 A10.0 H10.0 mm
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	---
Initial Position	L0.0 A10.0 H10.0
L	0.0 mm
A	10.0 mm
H	10.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Transversal

Geometry - Saturation

Fat suppr.	Fat sat.
Fat sat. mode	Weak
Special sat.	None

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	FIX
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Adaptive Combine
Save uncombined	Off
Matrix Optimization	Off
AutoAlign	---
Coil Select Mode	Off - All

System - Adjustments

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

Sequence - Special

Dummy Scans	1
Reverse Phase Encoding	Off

System - Adjust Volume

Position	L0.0 A10.0 H10.0 mm
Orientation	Transversal
Rotation	0.00 deg
A >> P	216 mm
R >> L	216 mm
F >> H	144 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
--------------	----------

System - Tx/Rx

Frequency 1H	123.255216 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	2.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	7030 ms
Concatenations	1

Inline - Common

Subtract	Off
Measurements	1
StdDev	Off
Save original images	On

Inline - MIP

MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

Inline - Composing

Inline Composing	Off
Distortion Corr.	Off

Sequence - Part 1

Introduction	Off
Multi-slice mode	Interleaved
Free echo spacing	On
Echo spacing	0.51 ms
Bandwidth	2778 Hz/Px

Sequence - Part 2

EPI factor	90
RF pulse type	Normal
Gradient mode	Performance

\\USER\CAMRIS\Satterthwaite\SATTERTHWAITE_ABCD_32ch\ABCD_fmri_rest

TA: 5:11 PM: FIX Voxel size: 2.4x2.4x2.4 mmPAT: 6 Rel. SNR: 1.00 : epfid

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	On
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	On
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	On
Wait for user to start	On
Start measurements	Single measurement

Routine

Slice group	1
Slices	60
Dist. factor	0 %
Position	L0.0 A10.0 H10.0 mm
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	---
Phase oversampling	0 %
FoV read	216 mm
FoV phase	100.0 %
Slice thickness	2.4 mm
TR	800 ms
TE	30.0 ms
Averages	1
Concatenations	1
Filter	None
Coil elements	HEA;HEP

Contrast - Common

TR	800 ms
TE	30.0 ms
MTC	Off
Flip angle	52 deg
Fat suppr.	Fat sat.

Contrast - Dynamic

Averages	1
Averaging mode	Long term
Reconstruction	Magnitude
Measurements	383
Delay in TR	0 ms
Multiple series	Off

Resolution - Common

FoV read	216 mm
FoV phase	100.0 %
Slice thickness	2.4 mm
Base resolution	90
Phase resolution	100 %
Phase partial Fourier	Off
Interpolation	Off

Resolution - iPAT

Accel. mode	Slice accel.
Accel. factor PE	1
Ref. lines PE	22

Resolution - iPAT

Accel. factor slice	6
Reference scan mode	EPI/separate

Resolution - Filter Image

Distortion Corr.	Off
Prescan Normalize	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off
Hamming	Off

Geometry - Common

Slice group	1
Slices	60
Dist. factor	0 %
Position	L0.0 A10.0 H10.0 mm
Orientation	Transversal
Phase enc. dir.	A >> P
FoV read	216 mm
FoV phase	100.0 %
Slice thickness	2.4 mm
TR	800 ms
Multi-slice mode	Interleaved
Series	Interleaved
Concatenations	1

Geometry - AutoAlign

Slice group	1
Position	L0.0 A10.0 H10.0 mm
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	---
Initial Position	L0.0 A10.0 H10.0
L	0.0 mm
A	10.0 mm
H	10.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Transversal

Geometry - Saturation

Fat suppr.	Fat sat.
Special sat.	None

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	FIX
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Sum of Squares
Matrix Optimization	Off

System - Miscellaneous

AutoAlign	---
Coil Select Mode	Off - All

System - Adjustments

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	L0.0 A10.0 H10.0 mm
Orientation	Transversal
Rotation	0.00 deg
A >> P	216 mm
R >> L	216 mm
F >> H	144 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Standard

System - Tx/Rx

Frequency 1H	123.255216 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	Pulse/Trigger
Average cycle	902 ± 1 ms
Average cycle	907 ± 1 ms
Acquisition window	833 ms
Trigger pulse	1
Trigger delay	0 ms
TR	800 ms
Concatenations	1
Phases	1

BOLD

GLM Statistics	Off
Dynamic t-maps	Off
Ignore meas. at start	0
Ignore after transition	0
Model transition states	Off
Temp. highpass filter	Off
Threshold	4.00
Paradigm size	3
Meas[1]	Baseline
Meas[2]	Baseline
Meas[3]	Active
Motion correction	Off
Spatial filter	Off
Measurements	383
Delay in TR	0 ms
Multiple series	Off

Sequence - Part 1

Introduction	Off
Multi-slice mode	Interleaved
Free echo spacing	Off
Echo spacing	0.51 ms
Bandwidth	2778 Hz/Px

Sequence - Part 2

EPI factor	90
RF pulse type	Normal
Gradient mode	Fast
Excitation	Standard

Sequence - pTX Pulses**Sequence - Special**

Dummy Scans	0
K-space streaming	None
Physio recording	Continuous
Reverse Phase Encoding	On

\\USER\CAMRIS\Satterthwaite\SATTERTHWAITE_ABCD_32ch\ABCD_fmRI_fracnback

TA: 8:04 PM: FIX Voxel size: 2.4x2.4x2.4 mmPAT: 3 Rel. SNR: 1.00 : epfid

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	On
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	On
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	On
Wait for user to start	On
Start measurements	Single measurement

Routine

Slice group	1
Slices	60
Dist. factor	0 %
Position	L0.0 A10.0 H10.0 mm
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	---
Phase oversampling	0 %
FoV read	216 mm
FoV phase	100.0 %
Slice thickness	2.4 mm
TR	2000 ms
TE	30.0 ms
Averages	1
Concatenations	1
Filter	None
Coil elements	HEA;HEP

Contrast - Common

TR	2000 ms
TE	30.0 ms
MTC	Off
Flip angle	75 deg
Fat suppr.	Fat sat.

Contrast - Dynamic

Averages	1
Averaging mode	Long term
Reconstruction	Magnitude
Measurements	240
Delay in TR	0 ms
Multiple series	Off

Resolution - Common

FoV read	216 mm
FoV phase	100.0 %
Slice thickness	2.4 mm
Base resolution	90
Phase resolution	100 %
Phase partial Fourier	Off
Interpolation	Off

Resolution - iPAT

Accel. mode	Slice accel.
Accel. factor PE	1
Ref. lines PE	12

Resolution - iPAT

Accel. factor slice	3
Reference scan mode	EPI/separate

Resolution - Filter Image

Distortion Corr.	Off
Prescan Normalize	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off
Hamming	Off

Geometry - Common

Slice group	1
Slices	60
Dist. factor	0 %
Position	L0.0 A10.0 H10.0 mm
Orientation	Transversal
Phase enc. dir.	A >> P
FoV read	216 mm
FoV phase	100.0 %
Slice thickness	2.4 mm
TR	2000 ms
Multi-slice mode	Interleaved
Series	Interleaved
Concatenations	1

Geometry - AutoAlign

Slice group	1
Position	L0.0 A10.0 H10.0 mm
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	---
Initial Position	L0.0 A10.0 H10.0
L	0.0 mm
A	10.0 mm
H	10.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Transversal

Geometry - Saturation

Fat suppr.	Fat sat.
Special sat.	None

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	FIX
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Sum of Squares
Matrix Optimization	Off

System - Miscellaneous

AutoAlign	---
Coil Select Mode	Off - All

System - Adjustments

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	L0.0 A10.0 H10.0 mm
Orientation	Transversal
Rotation	0.00 deg
A >> P	216 mm
R >> L	216 mm
F >> H	144 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Standard

System - Tx/Rx

Frequency 1H	123.255216 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	2000 ms
Concatenations	1

BOLD

GLM Statistics	Off
Dynamic t-maps	Off
Ignore meas. at start	0
Ignore after transition	0
Model transition states	Off
Temp. highpass filter	Off
Threshold	4.00
Paradigm size	3
Meas[1]	Baseline
Meas[2]	Baseline
Meas[3]	Active
Motion correction	Off
Spatial filter	Off
Measurements	240
Delay in TR	0 ms
Multiple series	Off

Sequence - Part 1

Introduction	Off
Multi-slice mode	Interleaved
Free echo spacing	Off
Echo spacing	0.51 ms
Bandwidth	2778 Hz/Px

Sequence - Part 2

EPI factor	90
RF pulse type	Normal
Gradient mode	Fast
Excitation	Standard

Sequence - pTX Pulses**Sequence - Special**

Dummy Scans	0
K-space streaming	None
Physio recording	Continuous
Reverse Phase Encoding	Off

\\USER\CAMRIS\Satterthwaite\SATTERTHWAITE_ABCD_32ch\ASL_3DSPIRAL_V20_GE

TA: 5:43 PM: REF Voxel size: 3.8×3.8×3.8 mmPAT: Off Rel. SNR: 1.00 : Sp

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	On
Start measurements	Single measurement

Routine

Slab group	1
Slabs	1
Dist. factor	20 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	---
Slice oversampling	12.5 %
Slices per slab	32
FoV read	240 mm
FoV phase	100.0 %
Slice thickness	3.75 mm
TR	4000.0 ms
TE	10.0 ms
Averages	1
Concatenations	1
Filter	None
Coil elements	HC1-7

Contrast - Common

TR	4000.0 ms
TE	10.0 ms
TD	0 ms
MTC	Off
TI	150 ms
Flip angle	90 deg
Fat suppr.	Fat sat.
Water suppr.	None

Contrast - Dynamic

Averages	1
Averaging mode	Short term
Reconstruction	Magnitude
Measurements	80
Pause after meas.	0.0 s
Multiple series	Off

Resolution - Common

FoV read	240 mm
FoV phase	100.0 %
Slice thickness	3.75 mm
Base resolution	64
Slice resolution	100 %
Slice partial Fourier	5/8
Interpolation	Off

Resolution - iPAT

PAT mode	None
----------	------

Resolution - Filter Image

Image Filter	Off
Distortion Corr.	Off
Prescan Normalize	Off
Normalize	Off
B1 filter	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off

Geometry - Common

Slab group	1
Slabs	1
Dist. factor	20 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
Slice oversampling	12.5 %
Slices per slab	32
FoV read	240 mm
FoV phase	100.0 %
Slice thickness	3.75 mm
TR	4000.0 ms
Multi-slice mode	Interleaved
Series	Ascending
Concatenations	1

Geometry - AutoAlign

Slab group	1
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	---
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Transversal

Geometry - Saturation

Sat. region	1
Thickness	10 mm
Position	Isocenter
Orientation	Transversal
Water suppr.	None
Special sat.	None

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	REF
Table position	H

System - Miscellaneous

Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Adaptive Combine
Save uncombined	Off
Matrix Optimization	Off
AutoAlign	---
Coil Select Mode	On - Coil Memory

System - Adjustments

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
A >> P	240 mm
R >> L	240 mm
F >> H	120 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
--------------	----------

System - Tx/Rx

Frequency 1H	123.255216 MHz
Correction factor	1
Gain	Low
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	4000.0 ms
Concatenations	1

Inline - Common

Measurements	80
--------------	----

Inline - Composing

Inline Composing	Off
Distortion Corr.	Off

Sequence - Part 1

Introduction	Off
Dimension	3D
Contrasts	1
Multi-slice mode	Interleaved
Bandwidth	400 Hz/Px

Sequence - Part 2

Gradient mode	Normal
RF spoiling	Off

Sequence - Special

PreSat	On
M0 Scan	On
Label Amplitude ON	On
Log physio	On
Inferior Saturation	On
Unbalanced pCASL	Off
[1] RF Flip Angle	28.7 °
[2] RF Gap	380 us
[3] RF Duration	500 us
[4] RF Grad Ramp Time	60 us
[5] Label Plane Offset	105 mm
[6] Gaverage (x10)	10 mT/m
[7] Gmax / Gavg	8
[8] Labeling Duration	1500000 us
[9] Post Label Delay	1500000 us
ASL Mode	Label-Control
Type BS pulse	C-FOCI Pulse
Spiral Trajectory	Spiral Out
[1] Sp Interleaves	2
[2] Sp Kz Segments	1
[3] Sp InPlane Segments	1
[4] Ice Dicom Scale	-500
[5] Ice M0 Scale	10
[6] Sp Max Grad	36 mT/m
[7] Sp Max Slew Rate	120 mT/m/ms
[8] Max Grad Rise Time	8.0 us/(mT/m)
Acceleration	Kz Acceleration (R=2)
[1] Partition R	3
[2] Partition RefBlocks	2
[3] In-Plane R 1	3
[4] In-Plane R 2	5
[5] In-Plane Area 1	30
[6] In-Plane Area 2	75
BS Parameters	Mu=12.6, beta=847.7
[3] No. BS Pulses	2
[4] Adiabatic Cond	20 (x10)

Sequence - Assistant

Mode	Off
------	-----

\\USER\CAMRIS\Satterthwaite\SATTERTHWAITE_ABCD_32ch\QSM_SWI_1.5mm

TA: 3:59 PM: FIX Voxel size: 1.5×1.5×1.5 mmPAT: 2 Rel. SNR: 1.00 : fl_r

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	Off
Start measurements	Single measurement

Routine

Slab group	1
Slabs	1
Dist. factor	20 %
Position	L0.0 A10.0 H10.0 mm
Orientation	Transversal
Phase enc. dir.	R >> L
AutoAlign	---
Phase oversampling	0 %
Slice oversampling	0.0 %
Slices per slab	96
FoV read	240 mm
FoV phase	75.0 %
Slice thickness	1.50 mm
TR	35.0 ms
TE 1	7.50 ms
TE 2	15.00 ms
TE 3	22.50 ms
TE 4	30.00 ms
Averages	1
Concatenations	1
Filter	Prescan Normalize
Coil elements	HEA;HEP

Contrast - Common

TR	35.0 ms
TE 1	7.50 ms
TE 2	15.00 ms
TE 3	22.50 ms
TE 4	30.00 ms
MTC	Off
Magn. preparation	None
Flip angle	15 deg
Fat suppr.	None
Water suppr.	None
SWI	Off

Contrast - Dynamic

Averages	1
Averaging mode	Short term
Reconstruction	Magn./Phase
Measurements	1
Multiple series	Each measurement

Resolution - Common

FoV read	240 mm
FoV phase	75.0 %

Resolution - Common

Slice thickness	1.50 mm
Base resolution	160
Phase resolution	100 %
Slice resolution	100 %
Phase partial Fourier	Off
Slice partial Fourier	Off
Interpolation	Off

Resolution - iPAT

PAT mode	GRAPPA
Accel. factor PE	2
Ref. lines PE	24
Accel. factor 3D	1
Reference scan mode	Integrated

Resolution - Filter Image

Image Filter	Off
Distortion Corr.	Off
Prescan Normalize	On
Unfiltered images	Off
Normalize	Off
B1 filter	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off

Geometry - Common

Slab group	1
Slabs	1
Dist. factor	20 %
Position	L0.0 A10.0 H10.0 mm
Orientation	Transversal
Phase enc. dir.	R >> L
Slice oversampling	0.0 %
Slices per slab	96
FoV read	240 mm
FoV phase	75.0 %
Slice thickness	1.50 mm
TR	35.0 ms
Multi-slice mode	Interleaved
Series	Interleaved
Concatenations	1

Geometry - AutoAlign

Slab group	1
Position	L0.0 A10.0 H10.0 mm
Orientation	Transversal
Phase enc. dir.	R >> L
AutoAlign	---
Initial Position	L0.0 A10.0 H10.0
L	0.0 mm
A	10.0 mm
H	10.0 mm
Initial Rotation	90.00 deg
Initial Orientation	Transversal

Geometry - Saturation

Saturation mode	Standard
Fat suppr.	None

Geometry - Saturation

Water suppr.	None
Special sat.	None

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	FIX
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Adaptive Combine
Save uncombined	Off
Matrix Optimization	Off
AutoAlign	---
Coil Select Mode	On - Coil Memory

System - Adjustments

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

! Position	L0.0 A10.0 H10.0 mm
! Orientation	Transversal
! Rotation	0.00 deg
! A >> P	216 mm
! R >> L	216 mm
! F >> H	144 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Slab-sel.

System - Tx/Rx

Frequency 1H	123.255216 MHz
Correction factor	1
Gain	Low
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	35.0 ms
Concatenations	1
Segments	1

Physio - Cardiac

Magn. preparation	None
Fat suppr.	None
Dark blood	Off

Physio - Cardiac

FoV read	240 mm
FoV phase	75.0 %
Phase resolution	100 %

Physio - PACE

Resp. control	Off
Concatenations	1

Inline - Common

Subtract	Off
Measurements	1
StdDev	Off
Liver registration	Off
Save original images	On

Inline - MIP

MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

Inline - Soft Tissue

Wash - In	Off
Wash - Out	Off
TTP	Off
PEI	Off
MIP - time	Off
Measurements	1

Inline - Composing

Inline Composing	Off
Distortion Corr.	Off

Sequence - Part 1

Introduction	Off
Dimension	3D
Elliptical scanning	Off
Phase stabilisation	Off
Asymmetric echo	Off
Contrasts	4
Flow comp. 1	Yes
Readout mode	Monopolar
Multi-slice mode	Interleaved
Bandwidth 1	200 Hz/Px
Bandwidth 2	200 Hz/Px
Bandwidth 3	200 Hz/Px
Bandwidth 4	200 Hz/Px

Sequence - Part 2

Segments	1
Acoustic noise reduction	None
RF pulse type	Fast
Gradient mode	Fast
Excitation	Slab-sel.
RF spoiling	On

Sequence - Assistant

Mode	Off
Allowed delay	0 s